



# Bonding Series

Strong Teeth,  
Strong Bond





**For a permanently safe bond of your restorations.**

## Correct Bonding

**Different materials make it necessary to use different bonders.**

**With the Schütz bonders you always have the right material at hand.**

With our bonding systems, you can safely join acrylics, composites, zirconium dioxide and metals. Select your bonding material depending on the indication:

- Extremely high bonding values
- Secure and durable bond
- Perfectly coordinated system
- Directly from the manufacturer

Visit us online.  
[www.schuetz-dental.de](http://www.schuetz-dental.de)



Sebond Family

### Schütz materials at a glance:

#### Liquid bonding agent for frameworks

- Sebond Smart (p. 3)
- Sebond Smart Zr (p. 3)
- Sebond Implant (p. 5)

#### Pasty bonders for frameworks

- Sebond Pink (light, dark) (p. 4)
- Sebond Bondopaker Dentin (p. 4)
- Sebond Grip (p. 5)

#### Bonders for natural teeth

- Capo Etch (chemical etching material)
- Capo Bond / Capo Bond Activator
- PrimeBond7 (7. generation bonder)

#### Luting composites

- Alphaslink
- Alphaslink Implant (p. 5) (also suitable for laboratory work)

#### Filling composites/veneering composites

- Capo Natural
- Capo Bulk Fill
- dialog Vario

For  
labs

For dental  
practices

# Sebond Smart

## The proven metal bonder

The metal bonder for a secure bond between acrylic or composite and metal. The bonder can be used universally on all conventional metals and with all conventional composite systems.



Sebond Smart 5 ml, art. no. 640066

### Your benefits:

- Extremely high bonding values
- No additional light polymerization necessary
- Odorless
- Universally applicable
- For a secure and durable bond
- Excellent biological compatibility


### Indications:

- Chemical adhesion of composite veneers
- Secure bond: acrylic/composite with metal

# Sebond Smart Zr

## The innovative zirconium dioxide bonder

The zirconium dioxide bonder for a secure bond between zirconium dioxide and composite. This bonder is universally applicable on all conventional zirconium dioxides and with all conventional composite systems.

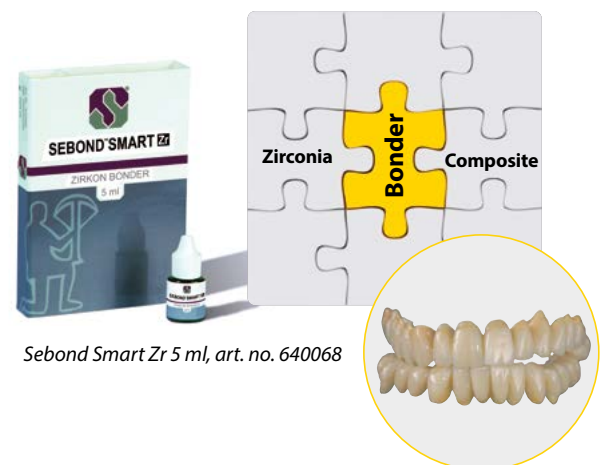


Sebond Smart Zr 5 ml, art. no. 640068

### Your benefits:

- No additional light polymerization necessary
- For a secure and durable bond
- Universally applicable

### Indication:

- Secure bond: zirconium dioxide/composite

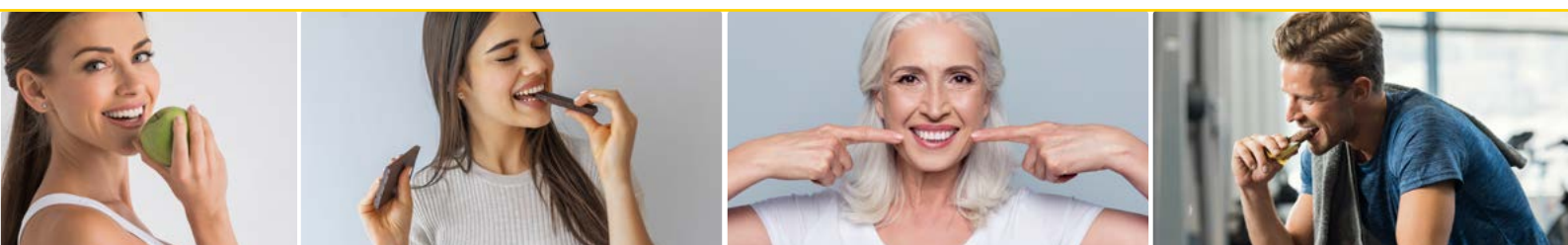





Visit us online.  
[www.schuetz-dental.de](http://www.schuetz-dental.de)

## Sebond Pink & Sebond Dentin


### Bonder and opaque in one material

The bond opaque with an extremely strong adhesion for a secure and lasting bond. Let the product's comfortable handling and the quality of its results convince you.

You plan on completing frameworks?  
Then this bonder is just right for you.

#### Your benefits:

- Opaque and bonder in one material
- Very high opacity
- The color adaption of the opaque to the acrylic color allows for excellent, consistent results
- Comfortable handling
- You will need two steps less in the procedure and you can do without the use of an additional metal bonder
- By means of color adaption, previously visible clamps "disappear" in the oral situation
- Save time and money
- Resistant to monomer and no streaks in the acrylic

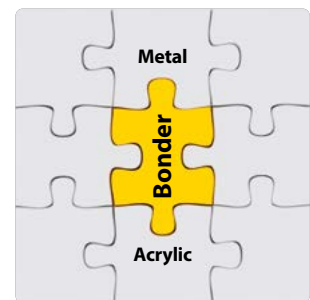


Sebond Pink light 5 g, art. no. 644090  
Sebond Pink dark 5 g, art. no. 644091



Sebond Dentin A3 2 g, art. no. 644094  
Sebond Dentin A4 2 g, art. no. 644095  
Sebond Dentin B1 2 g, art. no. 644096  
Sebond Dentin C3 2 g, art. no. 644098  
Sebond Dentin D2 2 g, art. no. 644099

#### Indications:

- For covering framework saddles and clamps

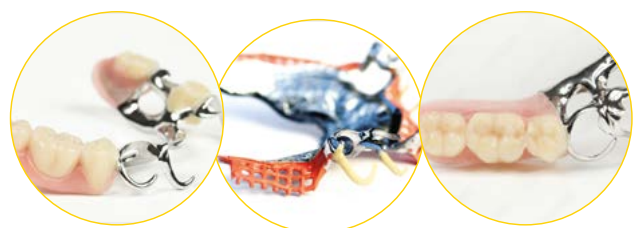


# Sebond Grip

## Universal Ground Coat

An additional mechanical bond for even better security.

### Your benefits:



- Simply apply Sebond Grip between the layers of metal bonder and paste opaque
- Easy flow behavior - thin compound layer between primer and metal
- Paste opaque does not need to be applied in the "wash" technique - this will save you time
- Low pigmentation = significantly faster polymerization
- Better bonding values and outstanding chemical bond to any paste opaques
- Excellent bond also in systems without metal bonder
- Masticatory forces and shear forces hardly stand a chance due to Sebond Grip

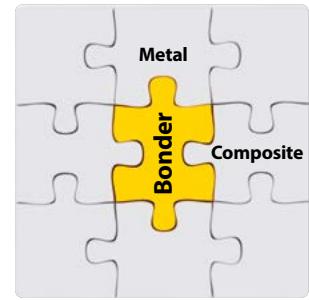

Sebond Grip 3 g, art. no. 644092

### Indications:

- For acrylic veneers

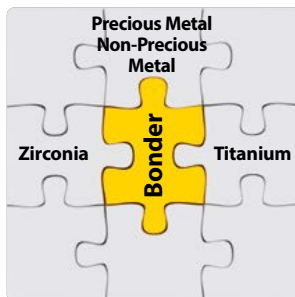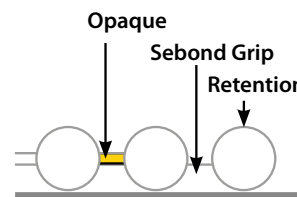

Sebond Implant 5 ml, art. no. 640075  
Alphalink Implant 2 x 4 g, art. no. 640076



# Sebond Implant & Alphalink Implant

## Strong combination

### Indications:

The Zirconium CEM was developed for:

1. Permanent fixation of indirect restorations made from metal (precious metal, non-precious metal, titanium), metal oxide ceramics (e.g. zirconium dioxide, aluminium oxide) or glass ceramics on implant abutments made from zirconium dioxide
2. Adhesive fixation of zirconium dioxide constructions on implant abutments made from zirconium dioxide or metal (e.g. titanium)
3. Adhesion of adhesive titanium bases with individual abutments made from zirconium dioxide

### The complete system consists of:

**Sebond Implant:** The bonding agent prepares the zirconium dioxide surfaces for a secure bond.

**Alphalink Implant:** The luting composite joins the abutment with your framework.

### Your benefits:

- Especially developed for bonding zirconium dioxide, with a high level of security
- Intraoral use = quick polymerization
- Use on a model = longer working range
- Excellent bonding values

The zirconium primer serves as bonding agent:



1. For the above listed indications
2. Between acrylics/composites and zirconium dioxide

*Implant fixed to an adhesive titanium base*

Pictures may vary.

# dialog Bonding Fluid

## – the universal acrylic bond


### Indications:

- Individualization, extension and repairs of light-curing veneering materials and acrylics

### Your benefits:

- Light-curing
- Forms a secure bonding layer between composite/composite and composite/acrylic
- Aesthetic individualizations (no dark margins)
- Very good handling
- By means of a special formula, even highly crosslinked acrylic teeth can be individualized safely and easily with this bonder

*Pictures may vary.*

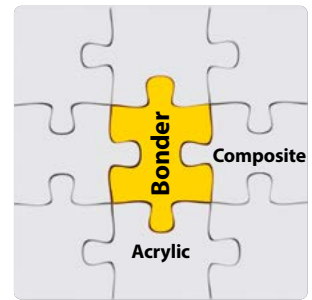

*dialog Bonding Fluid 7 ml,  
art. no. 643922*

## Courses & Events

Whether you are a dental technician, dentist or implantologist, consider continuing education for you and your team. Inspire your customers, motivate your employees and invest in your future today.

### This is what we offer:

- Exciting live previews
- Practical hands-on courses
- Individual trainings for you and your team
- From users for users

### Your benefits:

- The latest techniques and procedures
- From users for users
- Workshop character and professional exchange with your peers

**Have a look:  
[www.sdent.eu/events](http://www.sdent.eu/events)**

**Register for one of our many courses  
or plan your individual training or  
event with us:  
tel. +49 6003 814-362,  
e-mail: [export@schuetz-dental.de](mailto:export@schuetz-dental.de)**

# Matching products

## Futura Series

Find out more about the **outstanding, aesthetic restorations** you can create with the **color-stable acrylics** of the Futura-series.



## Comtray

Highly crosslinked, light-curing preshaped acrylic on **composite basis** which is **immediately ready for use**. For the production of individual trays, base plates and set-up aids.



art. no. 658035, white  
art. no. 658030, pink

## HS Cross-Liquid

Highly active, chemical abradant for **securely joining** crosslinked teeth to denture acrylics.

art. no. 659140



## Calibris Polishing Brushes

This polishing brush is ideal for polishing acrylic dentures. The integrated linen buff **intensifies** the polishing result and you will **save time**.

art. no. 660050



## FuturaSep Plus

One-component insulating agent on alginate basis. For secure insulation of plaster against acrylics.

art. no. 659121



## MicroClean

Dual denture cleaning system: mechanical and chemical cleaning all in one. **Especially gentle** cleaning procedure due to specially rounded cleaning needles.

art. no. 676410



## Futura Polish Polishing Paste

The **clean way** to polish acrylics. This polishing paste will help you achieve a **highly polished denture** in no time - and completely without wet pumice powder.

art. no. 659123



## Special cutter for acrylics

For denture acrylics, plaster, tray materials and die materials.

art. no. 681204







**We are here for you!**



**+49 (0) 6003 814-362**



**export@schuetz-dental.de**



**www.schuetz-dental.com**

## Schütz Dental Newsletter

**Register for our newsletter and be one step ahead.**

Benefit from exclusive offers and promotions for dental technology, dentistry and implantology which are only available through our newsletter. Never miss important information or events by Schütz Dental again. Always keep one decisive step ahead of your competitors.

**Register now: [sdent.eu/newsletterenglish](https://sdent.eu/newsletterenglish)**

**[www.schuetz-dental.com](https://www.schuetz-dental.com)**

**Visit us online!**

The details listed are examples. We will be glad to provide you with a specific offer.  
Errors and omissions excepted. Subject to goods being unsold and to changes without prior notice.



Schütz Dental GmbH • Dieselstr. 5-6 • 61191 Rosbach/Germany • Tel. +49 (0) 6003 814-362 • Fax +49 (0) 6003 814-907  
[www.schuetz-dental.com](https://www.schuetz-dental.com) • [export@schuetz-dental.de](mailto:export@schuetz-dental.de)

