# HS Cross-Liquid

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If there is anything in this instruction for use that you do not understand, please contact our customer service department before using the product.

As the manufacturer of this medical device, we inform our users and patients that all serious events occurring in connection with it must be reported to us (the manufacturers) as well as the relevant authorities in the Member State where the user and/or patient is resident.

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Methyl methacrylate, acetone

**Indication:**

Roughening agent for cross-linked teeth and acrylics.

**Contraindication/interaction with other agents:**

Do not use in case of a known allergy against one of the components.

**Application/processing:**

Thinly coat the mechanically roughened teeth with HS Cross-Liquid.  
No liquid „puddles“ must remain.  
Allow to evaporate for 2 - 3 min.  
Subsequently, apply acrylic.

**Storage:**

Store at 10 °C - 25 °C (50 °F - 77 °F).  
Close container well after each use.

**Hazard statement:**

May cause allergic skin reactions.

**Shelf-life:**

The maximum shelf-life is printed on the label of each packaging. Do not use after the expiration date.

**Side-effects:**

With proper use of this medical device, unwanted side-effects are extremely rare. Reactions of the immune system (e.g. allergies) or local discomfort, however, cannot be ruled out completely. Should you learn about unwanted side-effects – even if it is doubtful that the side-effect has been caused by our product – please kindly contact us.

**Contra-indications/interactions:**

If a patient has known hyper-sensitivities towards a component of this product, we recommend not to use it or to do so only under strict medical supervision. The dentist should consider known interactions and cross-reactions of the product with other materials already in the patient's mouth before using the product.

**Note:**

Please supply the attending physician with the above information, if this medical device is used to produce a special model. Please also consider the safety data sheets.

**Disposal:**

The liquid must be disposed of in hazardous waste.

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