FuturaIntensiv

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Prosthetic colourant

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Colore protesi

Tinte para prótesis

CE 0297

SCHÜTZ  **DENTAL**
Micerium Group

Schütz Dental GmbH · Dieselstr. 5-6 · 61191 Rosbach/Germany
Telefon: +49 (0) 6003 814-0 · Telefax: +49 (0) 6003 814-906
info@schuetz-dental.de · www.schuetz-dental.de

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If there is anything in this instruction for use that you do not understand, please contact our customer service department before using the product.

As the manufacturer of this medical device, we inform our users and patients that all serious events occurring in connection with it must be reported to us (the manufacturers) as well as the relevant authorities in the Member State where the user and/or patient is resident.

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Instructions for Use

Designation of purpose

These denture acrylics are intended for the fabrication of splints, partial dentures, full dentures and model dentures.

Product description

FuturaIntensiv powders are an addition to conventional denture acrylics and are used for color customization of dental restorations on PMMA basis.

Users

For application by dental technicians in a dental lab.

Composition

99% pearl polymer made of polymethyl methacrylate (PMMA), 1% pigments (all colors, besides white)

90% pearl polymer made of polymethyl methacrylate (PMMA), 10% pigments (color „white“)

Indications

- for color customization of dental restorations on PMMA basis
- characterization of existing dentures

Contraindications

Do not use if allergy to any of the components is known.

Cross reactions with other agents

Phenolic substances (e. g. Eugenol) inhibit polymerization. Therefore, do not use any materials that contain such substances.

Use / processing

FuturaIntensiv colors can be used with all denture acrylics of the Futura product line.

Color customization of dental restorations

Measure powder and liquid of the denture acrylic as usual. Add a small portion of Futura Intensiv to the measured liquid and stir briefly. Next, mix the measured powder with the liquid as usual. Process according to the instructions for use of the denture acrylic and finish the restoration.

Characterization of an existing denture

Roughen the surface of the denture to be characterized and wet it with monomer liquid. Prepare the required quantities of powder and liquid (cold curing acrylic). First add Futura Intensiv to the monomer liquid, then add the acrylic powder. Mix thoroughly. Let the mixture soak for a short time, then apply it to the roughened and monomer-coated surface of the denture.

Subsequent worksteps

Polymerize the characterized denture in a pressure pot for 15 min. (2 - 4 bar, 45° C / 113° F). Afterwards, finish as usual. Corrections can be carried out several times in the same way.

Note on processing

- Use colors only in small quantities, as they achieve a high color effect.
- The individual colors can be mixed with each other.

Note on storage and handling

- Storage temperature 10 - 25° C / 50 - 77° F.
- Close container tightly directly after use.

Shelf-life

The maximum shelf-life is printed on the label of each packaging. Do not use after the expiration date.

Note on side-effects

With proper use of this medical device, unwanted side-effects are extremely rare. Reactions of the immune system (e. g. allergies) or local discomfort, however, cannot be ruled out completely. All serious incidents occurring in connection with the use of this product must be reported to the specified manufacturer and the relevant competent authority.

Note on disposal

Residual quantities and packaging material must be disposed of in accordance with local and / or statutory regulations.

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